medela

symphony[®]
breast pump kit

Kit del extractor
de leche symphony[®]

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Table of Contents

1. Important Safeguards.....	4-5
2. Accessories	6-7
3. Getting Started.....	8
4. Cleaning Overview	9
5. Wash – Before first use and after each use*	10-11
6. Sanitize – Before first use and once per day*	12-13
7. Tubing and Breast Pump Care.....	14-15
7A. Tubing Care	14-15
7B. Cleaning and disinfecting the breast pump and pumping area	14-15
8. Assembly of Pump Kit	16-19
9. Breast Shield Sizing.....	20
10. Breast Milk Storage Guidelines	21
11. Troubleshooting	22-23
12. Supplemental Information.....	24
13. Warranty.....	25

IMPORTANT

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breast milk if bottles or components become damaged.

Intended Use

The Symphony® Breast pump is a powered breast pump to be used by lactating women to express and collect milk from their breast.

Product Description

Symphony is a hospital grade (multi-user) electric breast pump that includes 2-Phase Expression® technology and is capable of single and double pumping.

The warning symbol identifies all instructions that are important to safety. Failure to observe these instructions can lead to injury or damage to the breast pump. When used in conjunction with the following words, the warning symbols stand for:

WARNING Can lead to serious injury or death.

CAUTION Can lead to minor injury.

NOTE Can lead to material damage.

TIP Useful or important information that is not related to safety.

PLEASE SAVE THESE INSTRUCTIONS.

1. Important Safeguards

When using electrical devices, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

WARNING: To avoid fire, electrocution, or serious burns:

- Always unplug electrical product immediately after use.
- Do not use while bathing or showering.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not submerge pump unit into water or other liquid.
- Do not reach for a product if it has fallen into water or other liquid. Immediately unplug from electrical outlet and contact manufacturer.
- Do not spray or pour liquid directly onto the pump unit.
- Do not leave product unattended when plugged into an electrical outlet.
- Close supervision is necessary when this product is used near children.
- Use the product only for its intended use as described in this manual.
- Do not use attachments not recommended by the manufacturer, as this may cause hazards.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Keep power cord and all components away from heated surfaces.
- Never use while sleeping or when drowsy.
- Never drop or insert any object into any opening or tubing.
- Do not use outdoors.
- Keep appliance dry and do not place in direct sunlight.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Equipment not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- Separation from the mains is only assured through the disconnection of the power cord from the wall socket outlet.
- Position the device to ensure easy disconnection of the power cord from the supply mains.
- Use Medela original accessories only.
- There are no user-serviceable parts inside the Symphony. DO NOT ATTEMPT TO REPAIR YOURSELF. No modifications to the device are permitted.
- Always inspect power cord wires prior to use for damage or exposed wire. If damage is found, immediately discontinue use of power cord and call Medela Customer Service at 1-800-435-8316.
- Only use the power cord that comes with the Symphony. Make sure the voltage of the power cord is compatible with the power source.

**WARNING: To avoid health risk and reduce the risk of injury:**

- Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.
- Inspect the breast pump kit components prior to each use.
 - Do not use the components if they are damaged.
 - Do not use the components if dirt, mold or other contamination is visible.
- Inspect all appropriate pump components before each use.
- Clean and sanitize all parts that come in contact with your breast and breast milk prior to first use. Sterile kits are ready to use and do not require washing prior to first use.
- Do not modify any breast pump kit components.
- Wash all parts that come in contact with your breast and breast milk after every use.
- Do NOT continue pumping for more than 2 consecutive pumping sessions if no results are achieved.
- Do not thaw frozen breast milk in a microwave or in a pan of boiling water.
- If problems or pain occur, consult your lactation consultant or physician.

**CAUTION: Can lead to minor injury:**

- Pump only with the breast pump in an upright position.
- Do NOT shorten tubing between breast shield and pump.
- Make sure the voltage of the device is compatible with the power source.
- Plug the power cord into the breast pump first and then into the wall socket.
- Do NOT use abrasive cleaners/detergents when cleaning breast pump or breast pump parts.
- Never put breast pump motor in water or a sterilizer, as you can cause permanent damage to the breast pump.
- Do not attempt to remove the breast shield from your breast while pumping. Turn the breast pump off and break the seal between your breast and breast shield with your finger, then remove breast shield from your breast.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with your finger and remove the breast shield from your breast.
- Do not store wet or damp parts as mold may develop.
- Inspect the tubing for condensation or milk after each use. Wash or replace the tubing immediately after use, if milk is observed in the tubing.
- If tubing becomes moldy, discontinue use and replace tubing.
- Do not use the breast pump kit with a non-compatible breast pump.
- Do not use the breast pump kit for any other purpose than expressing breast milk.

2. Accessories

Lactina® to Symphony® Conversion Kit (contains (2) Symphony Protective Membranes and Membrane Caps)	67091
PVC Tubing - Symphony, Lactina®, Classic™	101034773
PersonalFit™ Connectors	87071
PersonalFit 21 mm Breast Shield	87072
PersonalFit 24 mm Breast Shield	87073
PersonalFit 27 mm Breast Shield	87274
PersonalFit 30 mm Breast Shield	87075
PersonalFit 36 mm Breast Shield	87084
Replacement Membranes	87088
Replacement Valves & Membranes	87089

Authentic Medela Spare Parts are made with the quality materials and workmanship you expect and trust, only from Medela.

Non-Medela spare parts can vary significantly in terms of product design, materials, and workmanship; all of which may affect the performance of your Medela breast pump.

Quality & performance
you can trust

3. Getting Started

! Note

It's important for you to do the following before using Symphony® for the first time:*

1. For models that include a rechargeable battery, fully charge the battery 12 hours uninterrupted.
2. Separate all parts.
3. Clean - see instructions:
 - Wash – section 5
 - Sanitize – section 6

! Note

*Sterile kits are ready to use and do not require washing prior to first use.

Breast shield assemblies come assembled. Separate all parts that will come into contact with the breast and breast milk before cleaning.

Parts to clean:

- Breast Shields
- Breast milk bottle/collection containers
- Lids
- Connectors
- White Membranes
- Yellow Valves

Kit components may vary from pictures in this instruction manual.

4. Cleaning Overview

	Breast pump kit & bottles/containers 	Tubing 
Before 1st use*	✓ wash ✓ sanitize	
After each use	✓ wash	
Once per day	✓ sanitize	
As needed		✓ wash only if milk is in the tubing or tubing is soiled

Detailed cleaning instructions on pages 10-15:

- Wash – page 10-11
- Sanitize – page 12-13
- Tubing care – pages 14-15

Note

*Sterile kits are ready to use and do not require washing prior to first use.

Warning

Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.

5. Wash – Before first use and after each use*

Supplies needed:

- Mild dish soap
- Clean dish cloth or soft brush
- Clean wash basin
- Drinking-quality water

Parts to wash:

- Breast shields
- Breast milk bottles
- Lids
- Connectors
- Yellow Valve Heads
- White Membranes

! Note

*Sterile kits are ready to use and do not require washing prior to first use.

! Warning

Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.

1

- Separate parts for washing.

! Caution

Separate and wash all parts that come in contact with breast and breast milk immediately after use to avoid dry up of breast milk residue and prevent growth of bacteria.

! Note

- Wash breast pump kit parts after each use.
- Take care not to damage parts of the pump kit during cleaning.
- **It is not necessary to wash tubing, membrane cap or protective membrane prior to first use.**
- **Only wash tubing if milk is present or tubing is soiled.**

4

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

! Note

Store the pump kit in a clean bag/container until next use.

Wash in sink with dedicated basin

2

Rinse all separated parts that came in contact with breast and breast milk in cool water in order to remove breast milk residue.

Alternatively, wash in dishwasher

1

- Wash all separated parts on top rack of dishwasher.
- Allow all breast pump parts to air dry in a clean area.
- Store dry parts in a clean, cool place when not in use.

Note

If the individual components of the pump kit are cleaned in a dishwasher, parts may be discolored by food pigments. This will not impact part function.

3

- Once rinsed, parts that have touched the breast or breast milk should be placed in a clean wash basin, dedicated to only cleaning these items. **Do not** place parts directly in the sink.
- Soak all separated parts in warm, soapy water for 5 minutes.
- Wash each part with a clean dishcloth or soft brush.
- Rinse all separated parts with clear water.

Caution

Only use drinking-quality tap or bottled water for cleaning.

You may also wash your kit parts by following the instructions on the Quick Clean™ Breast Milk Removal Soap bottle. (sold separately)

6. Sanitize – Before first use and once per day*

Supplies needed:

- Clean pot for boiling water
- Drinking-quality water
- Tongs

Parts to sanitize:

- Breast shields
- Breast milk bottles
- Lids
- Connectors
- Yellow Valve Heads
- White Membranes

! Note

Sanitize breast pump kit parts once daily.

! Warning

Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.

Medela pump kit parts can be sanitized by boiling or using the Medela Quick Clean™ Micro-Steam™ bag. (sold separately).

! Note

*Sterile kits are ready to use and do not require washing or sanitizing prior to first use.

- 3
- Fill a pot with enough water to cover all parts.
 - Bring water to a boil.
 - Place parts in boiling water for 10 minutes.

! Note


If you notice a white residue on your parts after boiling, you may have a high mineral content in your water. Remove residue by wiping parts with a clean washcloth and air dry. We recommend that you boil parts with distilled water to prevent substantial mineral build-up over time which may compromise your parts.

- Separate all parts that come in contact with breast milk.

- Wash hands thoroughly.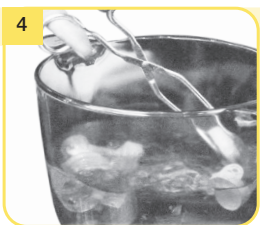

Allow water to cool and gently remove parts from water with tongs.

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Note

Store the pump kit in a clean bag/container until next use.

If sanitizing your pump kit parts with the Quick Clean Micro-Steam bag, separate all parts that come in contact with your breast or breast milk and follow the instructions on the Quick Clean Micro-Steam bag.

7. Tubing and Breast Pump Care

Supplies needed:

- Mild dish soap
- Clean dish towel
- Clean sink or bowl
- Drinking quality water
- Damp (not wet cloth)

Parts needed:

- Tubing
- Symphony® pump
- Membrane cap
- Protective membrane

7A. Tubing care

- Inspect tubing after each pumping session.

Caution

- Do not store wet or damp parts as mold may develop.
- Inspect the tubing for condensation or milk after each use. Wash or replace the tubing immediately after use if soiled or milk is observed in the tubing.
- If tubing becomes moldy, discontinue use and replace tubing. Replacement parts can be purchased at www.ShopMedela.com.

3

Remove tubing, protective membrane, and membrane cap. Clean protective membrane and membrane cap by following steps in **section 5**.

Note

- Tubing should be washed if milk is present or tubing is soiled.
- Cleaning tubing is not necessary if condensation present is from previous washings or atmospheric conditions.

Warning

Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.

4

Remove tubing from breast shield.

7

Use a damp (not wet) cloth to wipe the pump and membrane area.

Note


If there is condensation in the tubing: Shake water droplets out of the tubing and hang it up to dry or let the pump run with tubing attached for 1–2 minutes or until dry.


Turn off breast pump.

Unplug breast pump from power source.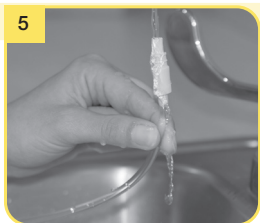

Rinse tubing in cool water to remove breast milk.

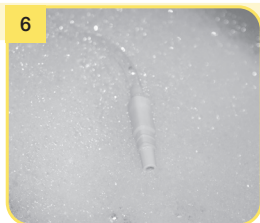- Wash tubing in warm soapy water.
- Rinse tubing with clear water.
- Hang to air dry.
- Or shake out any drops of water from the tubing and let the pump run with tubing attached until dry.

7B. Cleaning and disinfecting the breast pump and pumping area

Wipe off breast pump and pumping area (ex: countertop or table) with a clean, damp (not wet) cloth. Water and mild dish soap or non-abrasive detergent may be used.

Wipe off the breast pump and pumping area with a clean, damp (not wet) cloth. Alcohol based disinfectant agent may be used to disinfect the breast

pump and pumping area (ex: countertop or table); follow the disinfectant agent's instructions for use.

Caution

Unplug breast pump before cleaning. Clean breast pump after every use and disinfect between users.

8. Assembly of Pump Kit

T Tip

Carry out all the steps with care and assemble the pump kit correctly for optimum performance.

Parts Needed:

- Tubing
- Symphony® pump
- Breast shields
- Breast milk bottles/containers
- Lids
- Connectors
- Yellow Valve Heads
- White Membranes
- Membrane cap
- Protective membrane

! Note

- Use Authentic Medela Spare Parts only. See **section 2** for details.
- Check pump kit components for wear or damage before use and replace if necessary.
- All components must be completely dry before use for proper performance and to prevent damage to the pump.

1

Caution

Wash hands thoroughly with soap and water before touching breast pump, kit and breasts. Avoid touching the inside of containers or lids.

4


Push yellow valve head onto inner ring of connector bottom.

7

Place the protective membrane onto a flat surface and carefully push the membrane cap down until it engages.

2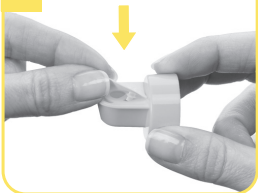

Carefully push the white membrane onto the yellow valve head until it engages and is laying flat.

3

Push the breast shield onto the connector.

5


Screw connector onto the bottle/container.

6

Insert the tubing into the corresponding opening in the connector.

8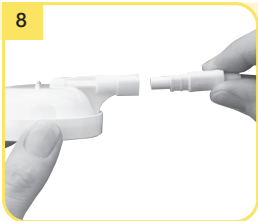

Insert the tubing adapter into the opening in the membrane cap.

Warning


Refer to breast pump instructions for use for important safeguards and operating information.

8. Assembly of Pump Kit (cont.)

Open the lid of the breast pump.

- ▶ Press the oval button on the top of the Symphony® and lift the handle.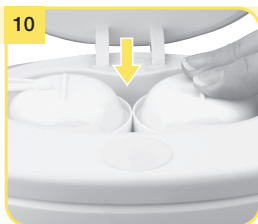

Position the membrane cap firmly on the breast pump membrane to form a seal. Center the tubing connector in the recess in the pump housing.

- ▶ Make sure that the membrane cap engages and lies flush to the surface.

Note

If single pumping, only seat one membrane cap.

Center breast shields over your nipples and start pumping. (Caution: pushing shields too firmly onto breasts may affect milk flow.)

To turn the pump on, press the power button. You will be starting in the Stimulation Phase. After 2 minutes, the pump will automatically change to the Expression Phase.*

Center the tubing connector in the recess in the pump housing.

- ▶ Make sure that the membrane cap engages and lies flush to the surface.

Close the lid.

Note

- **For step 12:**

Always close the lid when pumping. The lid keeps the membrane cap pushed down to prevent a loss of vacuum that could occur if the membrane cap came off.

If let-down occurs before the Expression Phase begins, press the let-down button to begin the Expression Phase.*

When your pumping session is over, store your breast milk and clean your kit. Refer to sections 3-6 for cleaning instructions. Refer to section 10 for breast milk storage guidelines.

*Pumping instructions are for Symphony Standard 2-Phase Expression® or when using the MAINTAIN program of Symphony PLUS® program card. Please refer to the Symphony Breast Pump Instructions for Use, item # 1907724, for more detailed pumping instructions.

9. Breast Shield Sizing

Caution

- While some discomfort may be felt when first using a breast pump, using a breast pump should not cause pain.
- If you are experiencing discomfort at the base of the nipple due to rubbing of your breast tissue with the breast shield tunnel, use of a lubricant such as Tender Care™ Lanolin may be beneficial.
- For assistance with correct breast shield sizing and comfort please visit www.medelabreastshields.com or see a lactation consultant / breastfeeding specialist.

Correctly fitting breast shields

Properly fitting breast shields will ensure comfort and efficient milk expression. To determine if you have the correct size, look at the following diagram:

- The nipple should be centered in the tunnel of the shield.
- The nipple should move freely without rubbing when the pump is turned on.
- Minimal or no part of your areola should be pulled into the tunnel of the breast shield.
- You should see a gentle, rhythmic motion in the breast with each cycle of the pump.
- After pumping, your breast should feel less full with no areas of hardness.

PersonalFit™ Breast Shields

21
mm

Item 87072

24
mm

Item 87073

27
mm

Item 87074


30
mm

Item 87075

36
mm


Item 87084

Too Small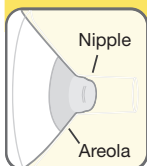


- Nipple rubs along side of tunnel
- Try a larger size

Correct Fit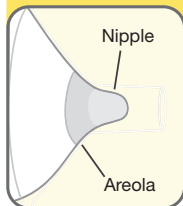

- Nipple is centered and moves freely

Too Large

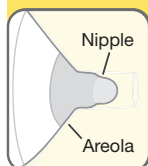- Nipple and excessive areola are pulled into the tunnel
- Try a smaller size

PersonalFit™ Breast Shields are available at many local retailers or visit www.shopmedela.com.

10. Breast Milk Storage Guidelines

10. Storing breast milk

Storage Guidelines for Freshly Expressed Breast Milk (For Healthy Term Babies)			
Room Temperature 60°F to 77°F (16° C to 25° C)	Refrigerator 39°F (4°C) or colder	Freezer 0°F (-18°C) or colder	Breast Milk Thawed in the Refrigerator
Up to 4 hours is best *Up to 6 hours for milk expressed under very clean conditions	Up to 3 days is best *Up to 5 days for milk expressed under very clean conditions	Up to 6 months is best *Up to 9 months for milk expressed under very clean conditions	At room temperature: Up to 2 hours Refrigerator: Up to 24 hours DO NOT REFREEZE

Referencias: www.BreastmilkGuidelines.com

*Guidelines for expressing milk under very clean conditions:

- Before expressing milk, wash your hands with soap and water or use an alcohol-based hand sanitizer.
- Pump parts, bottles and pumping area must be clean.
- Breasts and nipples do not need to be washed before pumping. These guidelines for storage and thawing of breast milk are a recommendation. Contact your lactation consultant or breastfeeding specialist for further information.

For additional breast milk collection and storage information, please visit:
www.BreastmilkGuidelines.com

11. Troubleshooting

If the breast pump is not running	Check whether electrical outlet power/battery is available. Check that the device is switched on. Check whether program card is inserted correctly in the card compartment on the back of the breast pump.
If there is low or no suction	<p>Assemble the pump kit in accordance with the instructions in Section 8.</p> <ul style="list-style-type: none">• Check whether all connections of the pump kit and breast pump are secure.• Make sure that the breast shield forms a complete seal around the breast.• The white membrane must lie flat against the yellow valve head.• Check that the yellow valve head and white membrane are clean and not damaged.• The ends of the tubing must be firmly connected to the connector and the opening in the membrane cap.• The protective membrane and membrane cap should be correctly assembled and the membrane cap must be firmly fitted to the breast pump membrane to form a seal.• The protective membrane must be intact.• Do not kink or pinch the tubing during expression.• All components must be completely dry. <p>Yellow Valve Head and Membrane Check</p> <ul style="list-style-type: none">• Remove and separate valve from the membrane.• Inspect the yellow valve head for chips or cracks. Inspect the white membrane for holes or tears. If flawed or damaged, replace the yellow valve head and/or white membrane prior to pumping.• Wash yellow valve head and white membrane per Cleaning Instructions in Sections 4-6.• Reassemble. Make sure the white membrane lays flat on the yellow valve head, and if it does not, replace the yellow valve head and/or white membrane prior to pumping. <p>Breast Shield and Breast Shield Connector Check</p> <ul style="list-style-type: none">• Inspect breast shield and breast shield connector to make sure there are no cracks and they securely attach to each other.• Make sure the breast shield forms a complete seal around the breast. <p>Other Checks</p> <p>Inspect the following to check for damage and secure connections.</p> <ul style="list-style-type: none">• Secure tubing adapter to the breast shield connector.• Test vacuum. If suction is not improved, contact Medela Customer Service at 1-800-435-8316. <p>Tubing Check</p> <ul style="list-style-type: none">• If damaged replace prior to pumping.• Inspect tubing for damage and secure connections.• Check that the ends of the tubing are firmly connected to the connector and the opening in the membrane cap.• If single pumping, make sure only one membrane cap is seated. <p>Take care not to kink or pinch tubing.</p>

<p>If there is an overflow</p>	<ol style="list-style-type: none"> 1. Place the pump set into the bottle holder. 2. Turn off the pump and unplug the power cord from the electrical outlet. 3. Use a damp (not wet) cloth to wipe the pump and membrane area. 4. If there is milk in the tubing: Remove tubing, protective membrane and membrane cap and clean in accordance with Section 7. Shake out any drops of water from the tubing and hang up the tubing to dry or let the pump run with tubing attached until dry. <p>If there is condensation in the tubing: Let the pump run with tubing attached until dry.</p> <ol style="list-style-type: none"> 5. When components are completely dry, reassemble breast pump and accessories set.
---------------------------------------	---

<p>Condensation or Milk in Tubing</p>	<ul style="list-style-type: none"> • Turn off the pump and unplug from power source. • Use a damp (not wet) cloth to wipe the pump. • Refer to cleaning instructions "Tubing Cleaning" Section 7. • If necessary, clean the pump kit. • When components are completely dry, reassemble breast pump and kit.
<p>No "Let-Down" or Breast milk Expression</p>	<ul style="list-style-type: none"> • Ensure your breast pump kit is assembled correctly and suction is available. • Relax and take a 10-15 minute break, if let down is not achieved after 2 consecutive pumping sessions. • Consult with a healthcare or lactation professional if expression does not occur.

If you have not resolved the problem with your breast pump or you have further questions, please contact Medela Customer Service at 1-800-435-8316 or fill out the contact us form at www.medelabreastfeedingus.com.

12. Supplemental Information

Supplemental Information

Following are some common breastfeeding related conditions. If you experience any of these symptoms, contact a healthcare professional or lactation professional.

	Symptom	Potential Causes
Engorgement	Breasts are hard and uncomfortable, possibly with reddened areas.	Milk not draining properly from breast. Milk can drain back into the tissue causing areas to swell and become tender.
Blocked/ Plugged Ducts	Area of breast will look reddened and may be tender to touch.	Milk not being drained from a specific duct. The area becomes “clogged” up and milk is then prevented from flowing.
Mastitis	It often follows engorgement but can occur suddenly. The initial symptoms are similar to flu – fatigue, headache and muscle aches, fever and localized breast tenderness. It can be in one or even both breasts and needs immediate treatment.	A bacterial infection in the breast tissue which is often accompanied by cracked nipples.

A Healthcare Professional or Lactation Professional can give you guidance in breastfeeding your baby and using a breast pump. Their guidance is valuable in terms of the lifetime of health benefits for both you and your baby. A listing of lactation professionals are available at www.medelalocator.com or calling 1-800-TELL-YOU (1-800-835-5968).

13. Warranty

This product is warranted by Medela LLC to the original purchaser to be free from defects in material and workmanship for the period of 90 days from the date of purchase. Warranty can only be claimed in the country of purchase. In the event of defect, Medela will replace this product, without charge for such replacement parts. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. THE LIABILITY OF MEDELA FOR BREACH OF ANY WRITTEN OR IMPLIED WARRANTY IS LIMITED TO REPAIR OR REPLACEMENT OF THIS PRODUCT, AND MEDELA WILL HAVE NO LIABILITY UNDER ANY CIRCUMSTANCES FOR ANY INCIDENTAL, PUNITIVE, OR CONSEQUENTIAL DAMAGES. SOME AREAS DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS OR DAMAGE LIMITATIONS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE OR IN YOUR COUNTRY.

Before you make a claim under this warranty, it may save time and expense to call Medela Customer Service (toll free) at 1-800-435-8316. You may also call this number for additional information concerning this warranty. If you wish to make a claim under this warranty, you must call first for authorization number; returns not accepted without an authorization number. Return this product to Medela with a return authorization number received from Medela Customer Service, prepaid, together with your dated bill of sale or other proof of purchase.

Customer Service toll free number: 1-800-435-8316

Medela LLC – Returns, Door 4501
1101 Corporate Dr.
McHenry, IL 60050
ATTENTION: RETURNS

Medela Canada Inc.
4160 Sladeview Cres., Unit #8
Mississauga, ON
Canada L5L 0A1

Call first for authorization number. Returns not accepted without an authorization number.

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Medela wordmark and logo, Symphony PLUS and Symphony are registered in the U.S. Patent and Trademark Office. PersonalFit, Mother's Milk, Everyday Amazing, Quick Clean, Tender Care and Micro-Steam are trademarks of Medela.
Le mot-symbole et le logo Medela, ainsi que Symphony PLUS et Symphony, sont déposés au bureau des brevets et des marques de commerce des États-Unis. PersonalFit, Mother's Milk, Everyday Amazing, Quick Clean, Tender Care and Micro-Steam are trademarks of Medela.
Medela Canada, Inc.